

# OralQ Oral Fluid Test

## STEP 1



Remove the collector from the sealed pouch and screw the collector cap counterclockwise to pull out the whole piece of collection stick with Sponge from the collection chamber (Step 1)

## STEP 2



Insert the sponge end of the collection stick into the mouth. Close mouth and gently chew the sponge for saliva excretion. Soak sponge into saliva in mouth and swab the inside of the mouth and tongue to collect oral fluid for a total of 1-3 minutes or until the sponge becomes completely soft and fully saturated with saliva. No hard spots should be felt on the sponge. (Step 2)

## STEP 3



Remove the sponge from the mouth. With gentle pressure, place the collection stick with saturated sponge into Collection Chamber and screw the Collector Cap clockwise to secure the cap. (Step 3)

## STEP 4



Place the saliva device on a clean and level surface. Unscrew the dispenser cap on the bottom end of the collector. (Step 4)

## STEP 5



Hold the collector upright and point the opening on the dispenser end toward specimen well of the test device. Use thumb and index finger to gently squeeze the pad in the Collection Chamber and dispense 1 drop of collected oral fluid into each specimen well. (Step 5)



The device and collector are both for single use.